
baboo®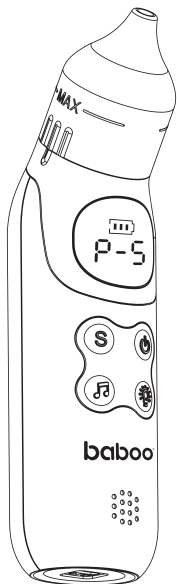

Nasal Aspirator **USER MANUAL** Model: ASP402

Software Version: V1.0. Manual Version: A1. Issue date: 2025.01

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
Product description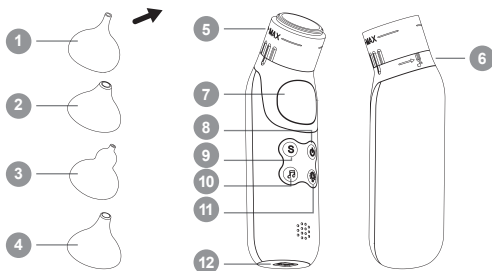

CONTENTS

1. General	3
2. Warning instructions and precautions	3
3. Contents	4
4. Product description	4
5. Before use	5
6. Instructions for use	6
7. Care and cleaning	8
8. Environmental impact	9
9. Technical specifications	9

1. General

The nasal aspirator is intended for the removal of nasal secretions and mucus from infants and children. It helps clear stuffy noses quickly and gently.

Please read all the instructions before using the nasal aspirator, especially this basic safety information.

2. Warning instructions and precautions

The warning signs and graphic symbols are intended to enable you to use the product safely and correctly and to prevent harm to you and others. The icons and meanings are as follows:

	Caution
	Date of manufacture
	Manufacturer
	Batch code
	Serial number
	Disposal in accordance with Directive 2012/19/EU (WEEE)
	CE conformity marking
	RoHS mark
	2 Protected against solid foreign objects of 12.5 mm Ø and greater 2 Protected against vertically falling water drops when enclosure tilted up to 15°
	Keep away from sunlight
	Keep dry

- This equipment is not suitable for use by persons (including children) with physical, sensory, or mental impairments, or those lacking experience and knowledge, unless they are supervised or instructed in the use of the equipment by a person responsible for their safety.
- Children should be supervised to ensure that they do not play with this product.
- The nasal aspirator is designed solely for cleaning or extracting nasal secretions.
- Do not use the device for purposes other than those stated in these instructions for use.
- Keep out of reach of children under 12 years of age.
- Do not expose the nasal aspirator to direct sunlight.
- This device is designed for domestic use only. Consult a doctor if the child is suffering from a nasal infection.
- Do not use any nozzle other than those supplied with this device.
- Do not use alcohol-based cleaners on the collection cup.
- Do not modify this equipment without the authorization of the manufacturer.
- If the container fills with liquid during suction, do not tilt it beyond 90 degrees, as the liquid may spill out.

3. Contents


1 pc. Electronic Nasal Aspirator, 1 pc. Snot Container, 4 pc. Silicone Tips, 1 pc. USB cable, 1 pc. User Manual

4. Product description

- | | |
|--------------------------------|---------------------------|
| 1. Funnel-shaped tip | 7. LED display |
| 2. Wide tip | 8. Start/Stop button |
| 3. Gourd-shaped tip | 9. Suction level adjuster |
| 4. Wide extended tip | 10. Music button |
| 5. Snot container | 11. Light button |
| 6. Reservoir twist lock/unlock | 12. Type-C charging port |

5. Before use


1. Charge the nasal aspirator battery for two hours. The battery display shows all three bars when charging is complete.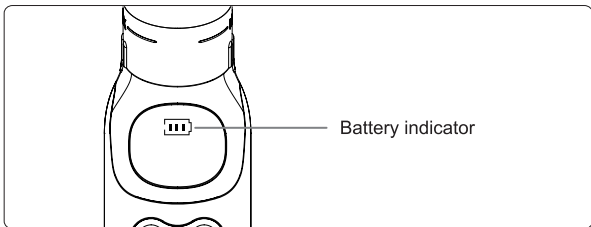


Note: For safety reasons, the nasal aspirator cannot be operated while charging.

In order to avoid battery damage due to low voltage for a long time, please charge the device every three months.

2. Disassemble the snot container and nose tips, clean and dry its parts.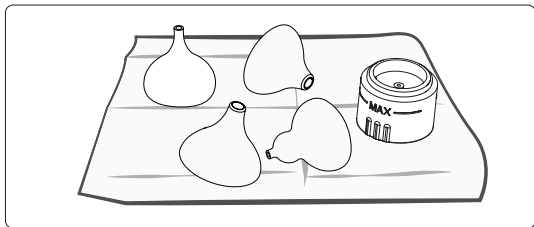


3. Attach the snot container. Only use clean and dry parts for assembly. Attach the silicone tip. Pick the size that fits your child's nose the best.

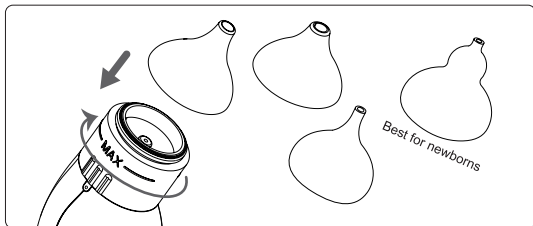6. Instructions for use


1. Press the Start/Stop button for 2 seconds to turn the nasal aspirator on.
2. Press the suction adjustment button to select the appropriate suction power.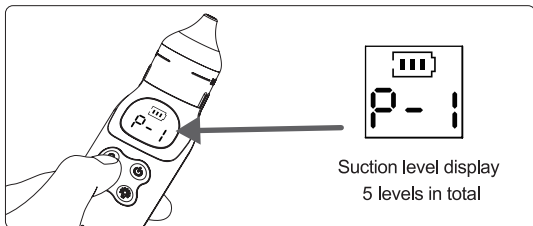


TIP: If the mucus is thin, you can use the device on a lower setting. If it is thick, you can use the device at a higher setting than usual.

NOTE: In case of swollen nasal mucosa or blocked nasal passages, use decongestant solution or nasal sprays 30 minutes before application.


3. Sit your child on your lap and hold them in your arms, or place them in an upright position with their back supported by a pillow. You may also seat them in an infant bouncer or infant seat.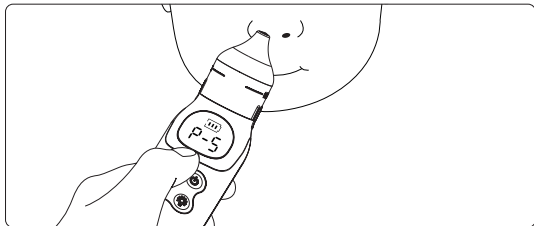

4. Press the Start/Stop button to pause operation. Press and hold the Start/Stop button for 2 seconds to turn the nasal aspirator off. The device will stop automatically after 5 minutes of inactivity.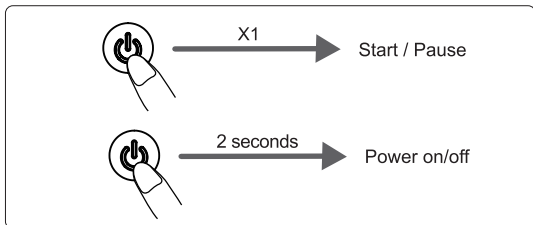

5. Watch the snot container fill with gunk. Do not exceed the MAX line in the snot container.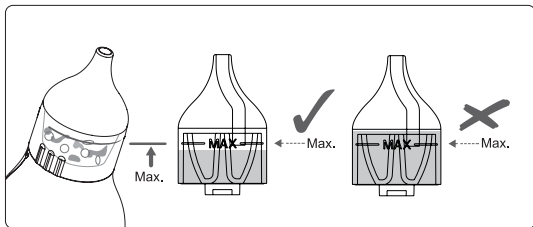

6. Note: Do not let nasal secretions overflow the MAX marker on the snot container.



If you want to distract your child, use the following function after turning on the power.

 Lights button:

x1 Color lights on
x1 Color lights off



 Music button:

x1 Music on
Long press for 2 seconds to change music.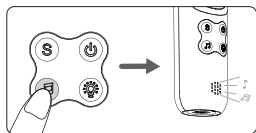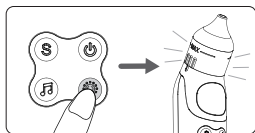

7. Care and cleaning

1. Twist  to lock/unlock snot container.
2. Gently pull off the tip.
3. Rinse or wash the snot container and tips. The water temperature must not exceed 100°C or the steam temperature.
4. Clean the motor unit with a damp cloth, and dry with a clean, dry towel.
Ensure that no liquid enters the interior of the nasal aspirator.

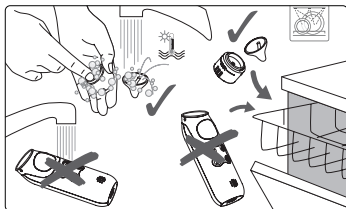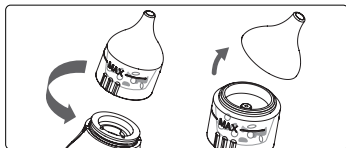	Boiling / how long	Rinse with boiling water	Steam sterilizer
Snot container	No	No	No
Tips	Yes. 5-10 min.	Yes, but with limited effectiveness.	Yes, according to the user manual of the sterilizer.

8. Environmental impact

Dispose of the device in accordance with your local recycling and waste disposal practices and requirements.

9. Technical specifications

Model	ASP402
Product	Nasal aspirator
Display	LED screen
Suction pressure	-55~-75kPa
Operation environment	Temperature: +10°C~+40°C Humidity: 15% ~95% R.H. non-condensing Pressure: 70kPa~106kPa
Transportation & storage environment	Temperature: -20°C~+55°C Humidity: ≤95% R.H. non-condensing Pressure: 70kPa~106kPa
Dimensions	167.6 x 47 x 53.7mm
Weight	150 g (with tip)
Power source	3.7 V 450 mAh Li-ion battery
Charging source	Input DC 5V/1 A
Anti-electric shock degree	Type BF
Mode of operation	Continuous operation
IP class	IP22

Expected lifetime of the device housing (electrical parts)	This product is covered by the Global Standard Warranty, which is valid from the date of purchase. The warranty period is two years in European countries and one year in all other regions worldwide.
Battery life	50 min
Overall Noise in dB	~ 45 dB (A) 0 m.
Class of Electric Shock Protection	BF
Exposure to Electromagnetic fields (EMF)	IEC 60601-1-2

Subject to change without notice.

Raw materials: Snot container: PC. Tips: Silicone. Body: ABS. Buttons: Silicone.

Discover more Baboo® products at www.baboo.baby. Baboo® is a registered trademark of Baby's Kingdom LTD, 10 John Street, London, WC1N 2EB, UK. Made in China. Importer in EU: UAB „Sentija“, Vilnius, Tuskulenu str. 13A-22, LT-09217, Lithuania, baboo@sentija.com.

 AOJ HEALTH Technology Co., Ltd. Room 202, Building 5, Shuhe Industrial Park, Sanwei Community, Hangcheng Street, Bao'an District, 518126 Shenzhen, PEOPLE'S REPUBLIC OF CHINA

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